



MARK IV ICU Bed


PN: 60-0019

The MARK IV ICU bed is manufactured to withstand 880 lbs. (400 kgs.) of resting force without material fatigue, and is extremely compact for easy storage and transportation.

The MARK IV ICU bed can be used in conjunction with mobile shelters for hospital / field hospital and humanitarian/ disaster relief applications.



INSTRUCTIONS FOR USE

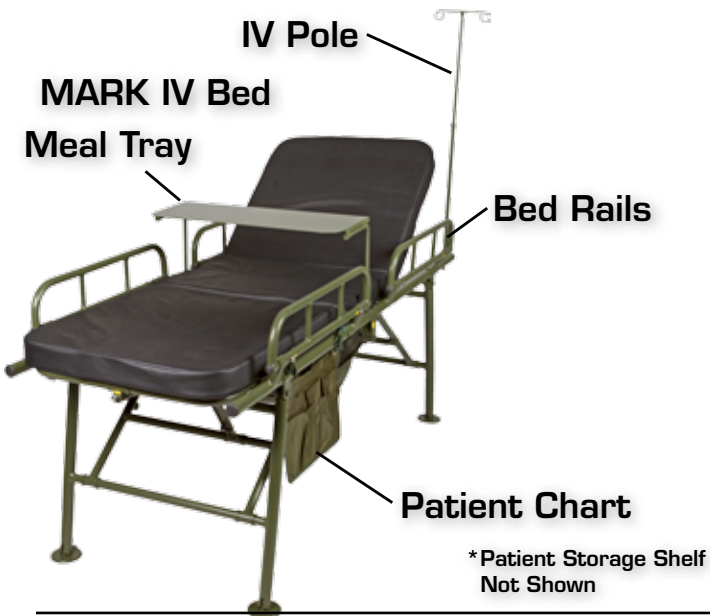

1



Prepare the bed in the flat state, as shown in the picture.

2



Open both sides of the bed fully to a 90° angle by using silver push button on either end.

3



Extend poles of bed to 180°




MARK IV ICU Bed

PN: 60-0019



INSTRUCTIONS FOR USE (Continued)

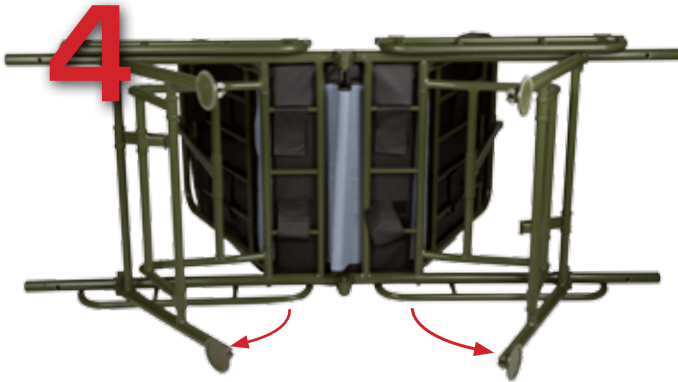

Fold out either side of the support legs until hinge is fully extended.



Place the bed upright and make sure the feet lay flat. Press button to adjust sides of bed up and down.



To raise or lower rail, press both buttons on sides of rail simultaneously.



To apply Patient Storage Shelf: Adhere hook and loop flaps around rails below bed.

8

At this point, the bed can be fitted with the Meal Tray, IV Pole, and Patient Chart.



NORTH AMERICAN RESCUE®

35 Tedwall Court • Greer, South Carolina 29650
Phone: (864) 675-9800 • Fax: (864) 675-9880
www.NARescue.com • 1-888-689-6277

Revised 12.27.12