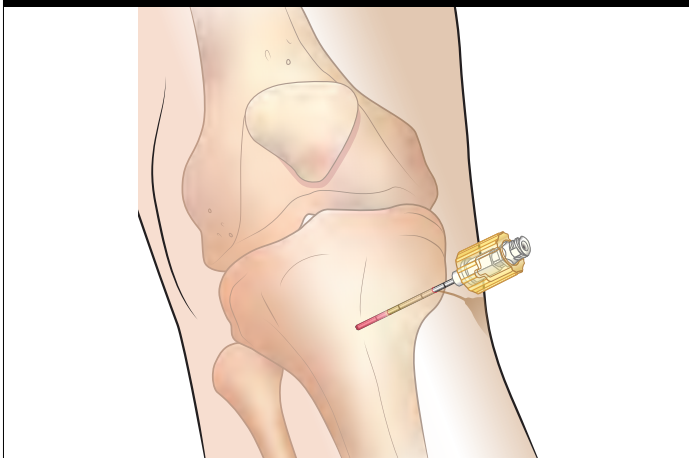
SAM[®] IO Stabilizer | Instructions for Use

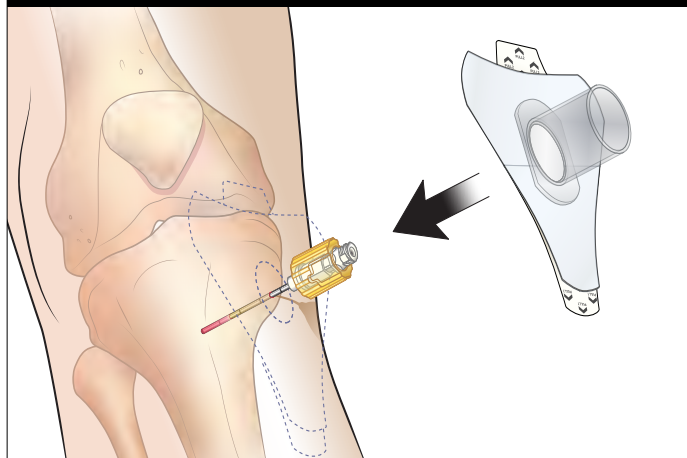
APPLICATION STEPS:

1


With catheter in position, consider use of Stabilizer.

2


Place sterile Stabilizer carefully over catheter and against skin.

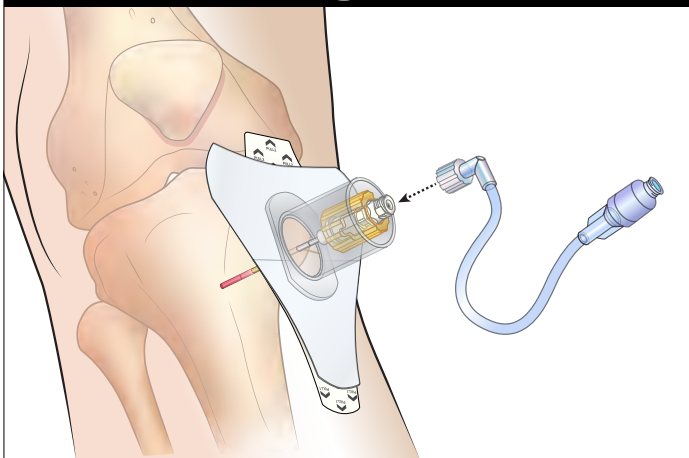3


Attach primed extension set to catheter.
KEY POINT: Extension set design permits single-handed tightening to catheter.

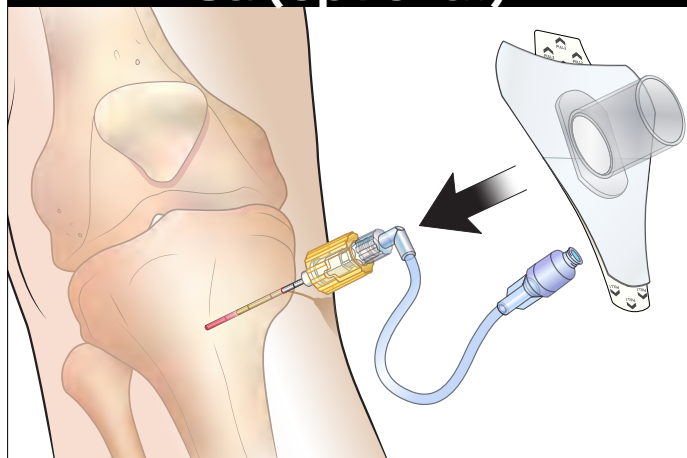
3a (optional)

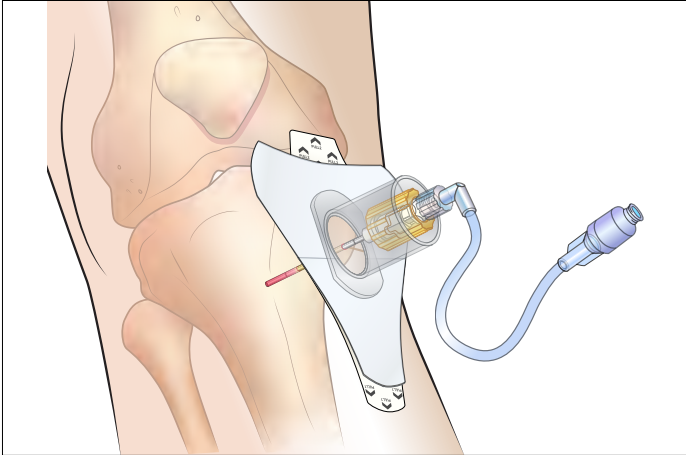
Optional Stabilizer Application Method:
With attached extension set in position gently feed extension set through Stabilizer cylinder. Guide Stabilizer over catheter, and against patient skin.

4

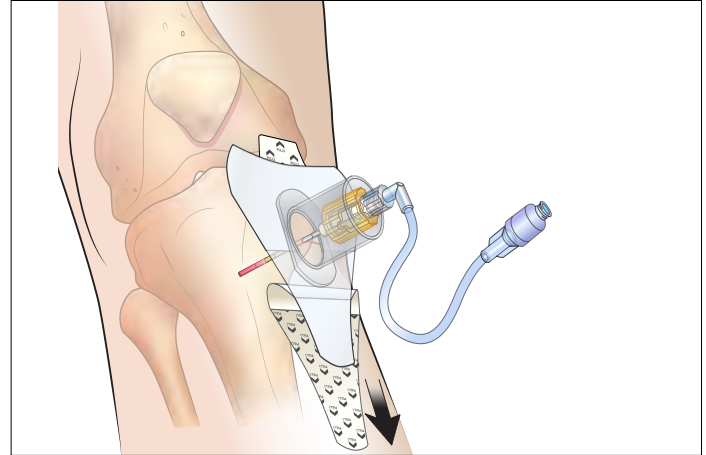
Ensure catheter is centered within Stabilizer.

5

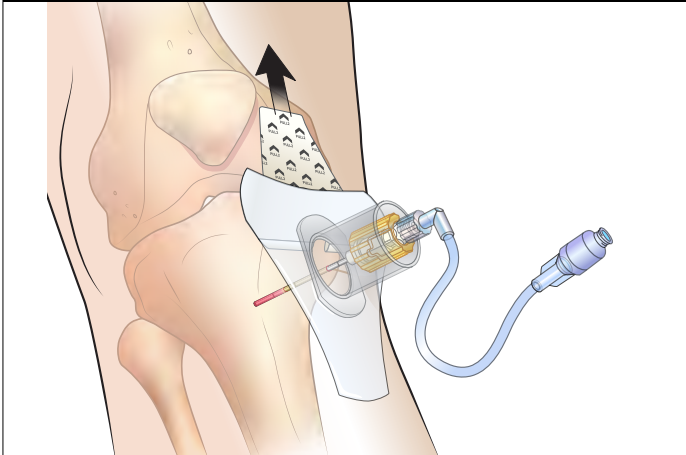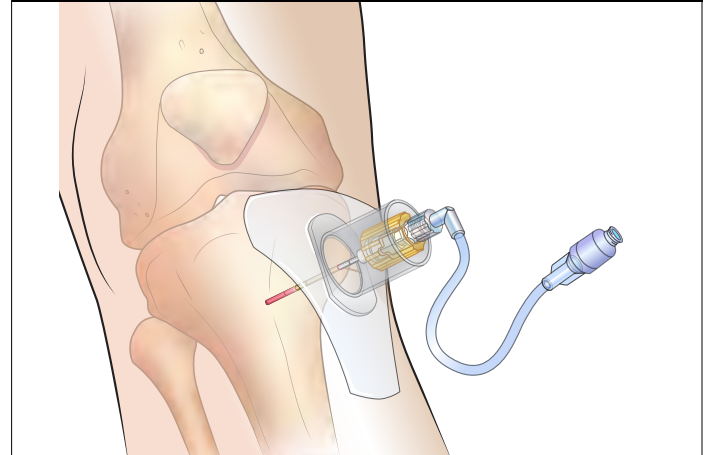Hold Stabilizer in position and gently pull Tab 1. By pulling Tab 1 you are removing adhesive backing. This will expose adhesive to patient skin. Gently press exposed adhesive against patient skin.


6

Hold Stabilizer in position and gently pull Tab 2. By pulling Tab 2 you are removing adhesive backing. This will expose adhesive to patient skin. Gently press exposed adhesive against patient skin.

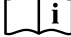7

With Stabilizer in position (and adhesive backing removed) gently smooth all exposed adhesive to patient skin.

SYMBOL GLOSSARY:

Indicates a medical device that should not be used if the package has been damaged or opened.	
Indicates a medical device that is intended for one use, or for use on a single patient during a single procedure.	
Indicates a medical device that has been sterilized using ethylene oxide.	
For prescription use only.	
Indicates the need for the user to consult the instructions for use.	
Not for Sternal use.	
Indicates a medical device that needs protection from light sources.	
Indicates a medical device that needs to be protected from moisture.	
Indicates a medical device that is not to be resterilized.	

 IO740-1P-EN - 1 Pack

 IO740-5P-EN - 5 Pack

SAM04041 A

GET TRAINED:
sammedical.com

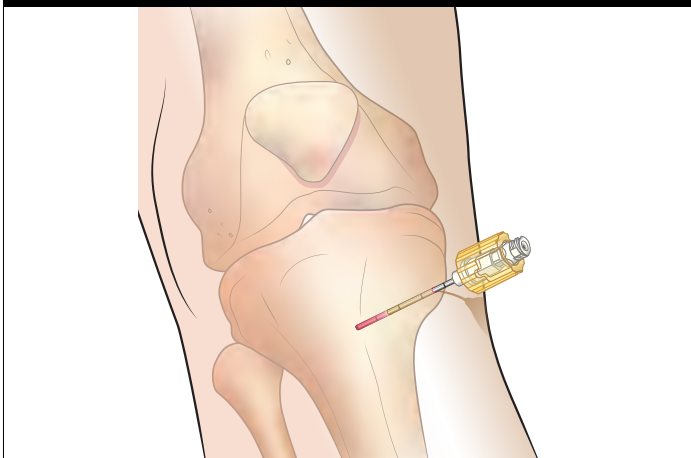
SAM Medical Products®
12200 SW Tualatin Rd, Ste 200
Tualatin, OR 97062 US

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