

# MCI-TKD

MASS CASUALTY INCIDENT  
- THROW KIT DISPENSER -

## Inner Spring Compartment Adjustment & Throw Kit Installation



1 Open both side pouches in order to access the cords and toggles used to adjust the Inner Spring Compartment of the dispenser. It is important to make equal adjustments to both sides.



2 Locate the Inner Spring Compartment flap at the bottom of the bag and pull the hook & loop tab to open.



3 Open the flap completely by pulling the flap down to unzip the sides of the flap.



4 Stand the bag on end and measure the distance from the TKD carrier to the opening of the compartment. Make the distance 6 inches to provide proper tension for dispensing kits.



5 Pull the elastic carrier cords in both side pouches an equal amount to shorten the distance of the carrier to the opening or give equal amounts of slack to lengthen.



6 Stand the bag on end and measure the distance from the TKD carrier to the opening of the compartment. Confirm that the distance from the carrier to the opening is 6 inches.



7 To lock the cord length, move the toggles by pressing the toggle button and sliding along the cord all the way to the corner of the pouch. Repeat both toggles in each side pouch.



8 For added security, tie a single knot in the cord behind each toggle.



9 The Throw Kits are ready to install once the cords in BOTH side pouches are adjusted to the proper lengths and secured to ensure that the TKD carrier distance is 6 inches from the opening.



10 Slide individual Throw Kits under the elastic strap and onto the TKD carrier stacking them flat on top of each other. Only load eight Throw Kits into the dispenser.



11 Once all eight Throw Kits are installed, gently push the loaded stack down on the carrier away from the opening approximately 1-2 inches.



12 Pull the Inner Spring Compartment flap over the opening and secure with the hook & loop.



13 Pull the zippered closures shut on each side. The adjustment and installation are complete.

