Emergency Trauma Dressing & Gauze Instructions For Use:

Intended Use:

Intended for external hemorrhage control, placed directly on or around a casualty wound.

Product Description:

The North American Rescue Bandage (Emergency Trauma Dressing or ETD[®]) and Gauze are single use, non-invasive dressings, and are intended to contact injured skin and act as a mechanical barrier to provide compression and absorb fluids. The intended users are typically military or EMS professional medics.

Instructions For Use:

1. Open exterior package and remove dressing.
2. Apply dressing on wound and ensure it is completely covered.
3. Wrap dressing around extremity, maintaining constant pressure, and fasten in place.
4. Once applied, check for distal circulation and adjust pressure of dressing as necessary to ensure proper circulation to the rest of the extremity.
5. Monitor casualty and dressing in accordance with your medical protocols, and transport to appropriate treatment facility.
6. Replace the dressing and obtain definitive wound care as soon as possible, ideally within 12 hours or as your medical protocols dictate.
7. Dispose of contaminated or soiled dressing in accordance with biohazard guidelines/protocols.

Warnings and Precautions:

- Contents sterile unless packaging open or damaged. If packaging is opened or damaged, DO NOT use device.
- Failure to adequately apply the bandage may allow for continued bleeding. Overtightening the bandage may cause a tourniquet effect, or tissue ischemia.
- These products contain cotton, polyester, and spandex. Do not use on casualties with known or suspected allergies to these materials.
- In the event of a malfunction, follow local protocols and report any serious incident to North American Rescue or authorized representative and the competent authority of the Member State.

Contraindications:

- Not indicated for pediatrics or pregnant women.

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MDI Europa GmbH