


ETD ABDOMINAL /STUMP

Emergency Trauma Dressing

Abdominal Evisceration INSTRUCTIONS FOR USE

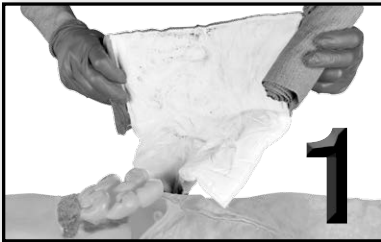


Unfold and Moisten
12 x 12 non-adherent,
sterile pad with Saline.



Place opened 12 x 12
pad directly over the
injury site and lay pad
flat, making sure it
covers the exposed
protruding tissue.



Wrap ETD around casualty's
torso working from the
lower abdomen to the
upper. Ensure that the entire
portion of the pad is covered
with the elastic wrap.

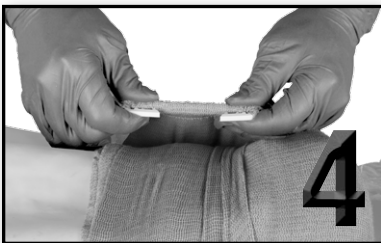

Secure the ETD on both
sides of the elastic with
the C-Clasp.

Treatment should focus on protecting the protruding segment of intestine or other organ from further damage. If the intestine or some of the other abdominal organs become dry, irreversible cell death can occur.

Eviscerated abdominal contents should be covered with a sterile dressing that has been moistened with saline (normal saline IV fluid can be used).

*These dressings should be **periodically remoistened** with saline to prevent them from drying out.*

Take efforts to prevent hypothermia.



NORTH AMERICAN RESCUE®
www.NARescue.com • 888.689.6277

PRODUCTS WITH A
MISSION®

REV042116