


# ETD ABDOMINAL /STUMP

Emergency Trauma Dressing

## Stump Application INSTRUCTIONS FOR USE

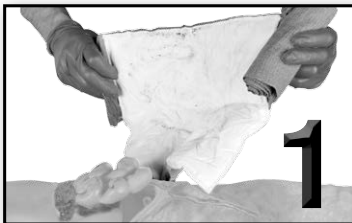

1 Before application of the Stump dressing: Ensure that a tourniquet has been properly applied and that the time of application has been noted.



2 Expose wound as required. Open Abdominal ETD and unfold 12" x 12" sterile, non adherent pad.



3 Place over distal end of stump, and Wrap tightly around limb, crossing bandage to cover distal portion of injury.



4 Wrap back across, crossing over to cover stump.



5 Pull tightly and secure using C-Clasp. Re-access for distal bleeding.

*If bleeding continues after an effective pressure dressing has been applied, consider additional tightening of the tourniquet or the use of a second tourniquet, side by side and proximal to the first to eliminate the distal pulse.*



**NORTH AMERICAN RESCUE®**  
www.NARescue.com • 888.689.6277

PRODUCTS WITH A  
**MISSION®**

REV042116